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Hay packs

Made of natural hay (therapyhay) without any additions. The hay packs exclusively contain therapyhay. It is used for haybathes and similar treatments.

The treatment must always be done according to doctor's orders.

These hay packs are steamed with water steam and then put onto the area which should be treated. During the treatment the body should be covered with a blanket. Treatment time is approx. 20 minutes.

Content: 40 pieces, approx. 40 x 30 cm,
approx. 320 g
Hay from unfertilised meadows

Storage: cool and dry

Best before: see label

Please contact us, if you need more information.

Field of application:

For heat therapy as an supporting action at: gastroenteritis, urogenital system, of the respiratory tract and the movement apparatus.

Contraindication:

Open injuries, acute rheumatic attacks, acute inflammation processes, high temperature.
Special caution on the application temperature at patients with obstruction of the thermoempfindlichkeit and patients with skin disease.

No treatment at patients with inclination to allergies.

Safety advice:

- Do not use hay whose shelf life has expired.
- Please keep away from children!
- Please drink plenty of liquids after overwarming treatments!
- Please note: Treatments with herbs may increase the photo-sensibility of the skin.
Hence, no direct sun and no solarium in the next 4 hours are allowed.

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Hersteller: Kurland GmbH, Kirchenwegstr. 5, 83404 Mitterfelden Airing